

LOW LEVEL LASER THERAPY

Hair Regrowth Therapy

SH650-1



SANHE



Low Level Laser

Hair Regrowth Therapy

LLLT ACCESSORIES
SH650-1




Hair & Skin Analyzer



Medicine Sprayer



Oxygen spray pen

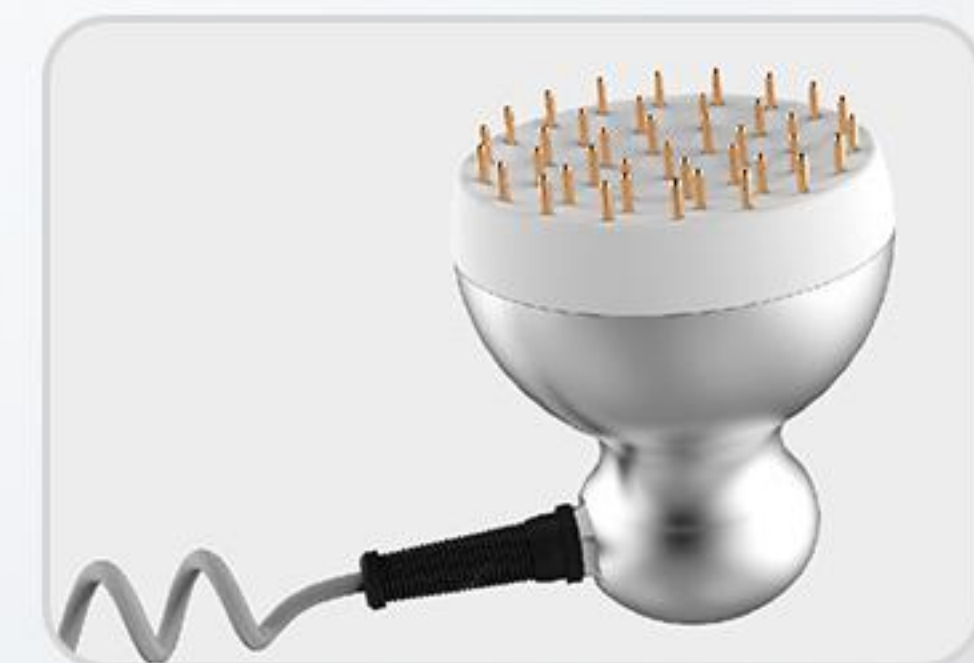

Microcurrent Massage Comb

Hair & Skin Analyzer

- ▶ Analyse the hair follicle and hair
- ▶ Store patients' treatment date.
- ▶ Befor & After, follow the result



Oxygen spray pen

The oxygen spray pen can generate saline spray by the airflow generated from the Hair Regrowth Therapy machine and do simple cleaning and disinfection treatment on the scalp.



Medicine Sprayer

Spray Hair Care Solution like Minoxidil solution into the hair follicles after cleaning the scalp, before treatment.



Microcurrent Massage Comb

There are Red light and Electrodes on the massage comb.

- ▶ The Microcurrent generated by the electrodes can accelerate the vessel open on the scalp.




Product Features:

Lasers and PDT lights;
Complementary Medicine: Minoxidil;
Easy To Operate;
Comfortable Treatment Process;
Remarkable Results.

Machine Size:1630*550*500 mm
(H*L*W)Package Size:1650*580*580mm
(H*L*W)Net Weight: 35kg**Gross**
Weight:55kg



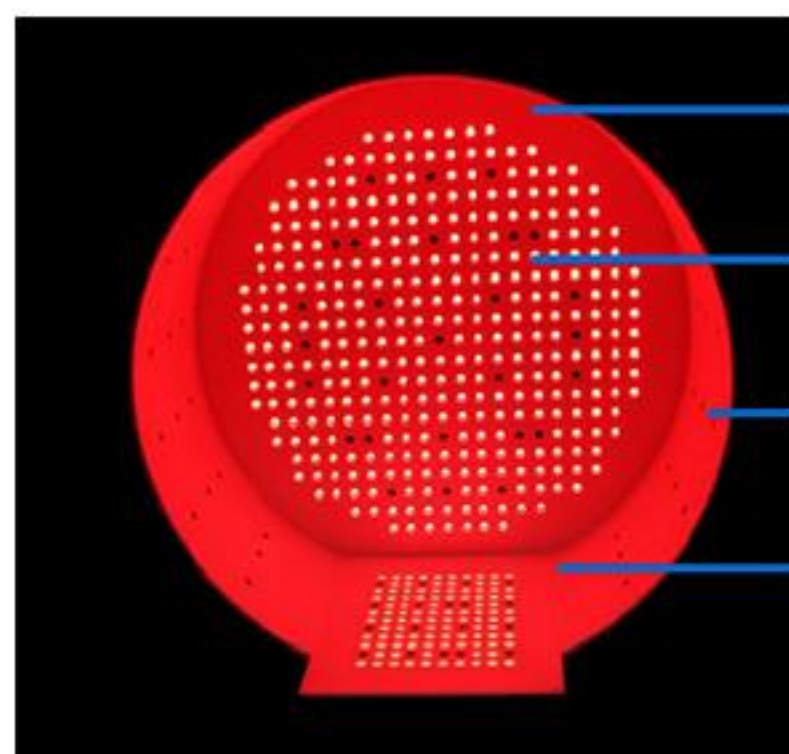

- 650nm Laser
- 808nm IR Laser
- Air spray
- 630nm red light
580nm Yellow Light
460nm Blue Light

Treatment Result:

- Increase Hair Density –40%-60%
- success rate in halting the progression of hair loss—85%





Red light

Make cell activity increased,
promote cells metabolism,
accelerate the blood circulation,
control oil secretion.



Yellow light

with wavelength 580 nm can ease
and treat sensitive skin effectively.



Blue light

kill hair pathogenic bacteria
effectively,kill the hidden fungi
and mites deep in the hair follicles.



MENU

SH650-1

Hair regrowth & Anti-hair loss system

LED	WORK TIME	<input type="text"/> m	SAVE	SINGLE
RED	ON TIME	<input type="text"/> s		DOUBLE
YELLOW	ON TIME	<input type="text"/> s		EXTRA
BLUE	ON TIME	<input type="text"/> s		READY

650nm laser	WORK TIME	<input type="text"/> m	↑	OK	
POWER	<input type="text"/>				↓
ON TIME	<input type="text"/> s	OFF TIME			

808nm laser	WORK TIME	<input type="text"/> m	Count:	
POWER	<input type="text"/>			
ON TIME	<input type="text"/> s	OFF TIME		



SPECIFICATION

Laser output power

650nm diode laser
808nm IR laser

LED output power

630nm Red cool light
460nm Blue Cool light
580nm Yellow Cool light

Treatment Time

0-60min

Pulsing time range

1-10s

Pulsing Delay range

0-10s

Input power

200w

LCD Screen

8inch multi color touch screen

Device Dimension


1630*550*500 mm

Package Dimension

1650*580*580 mm

Rated input voltage


AC230 10%,50Hz 1Hz
AC110V 10%,60Hz 1Hz (optional)

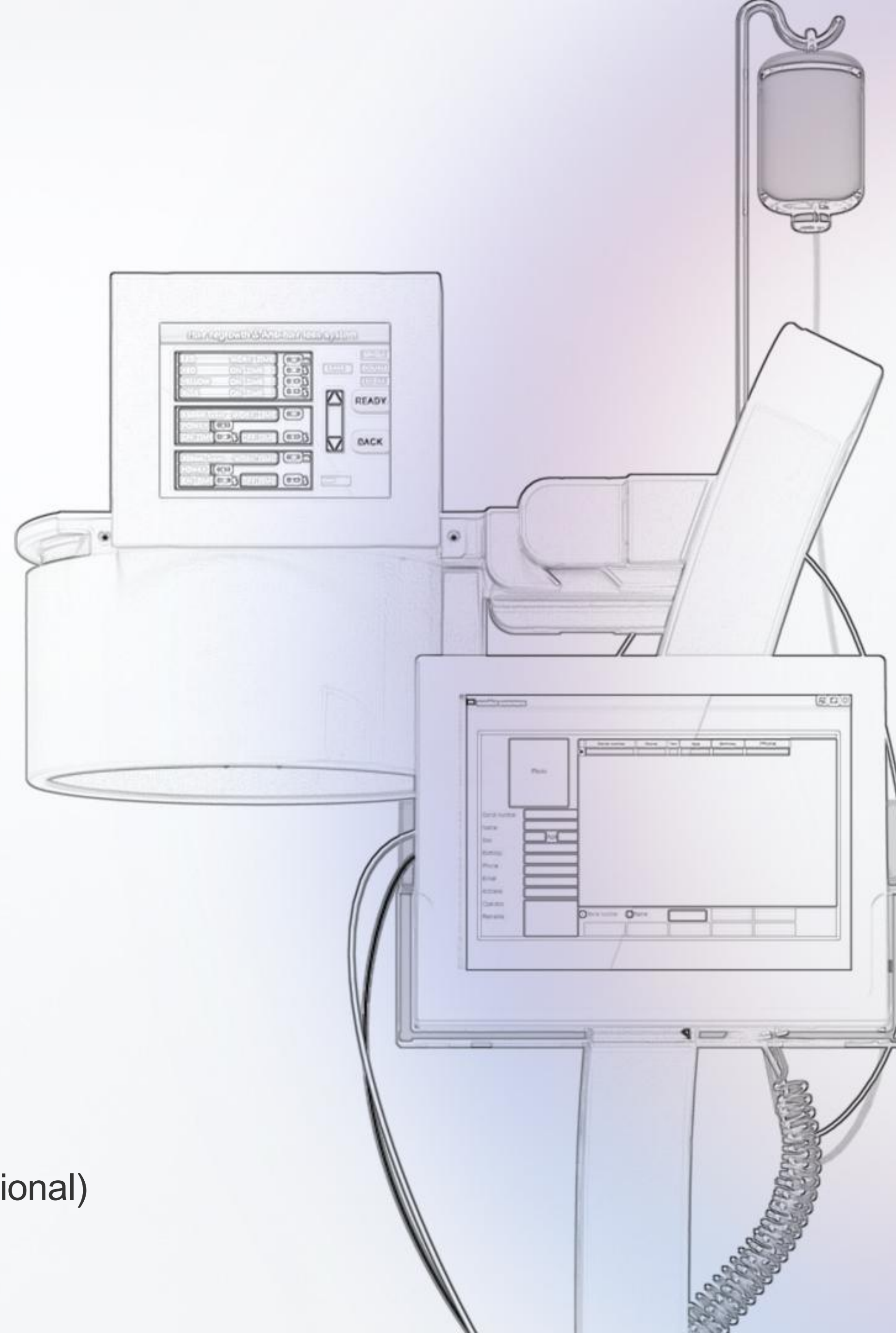

TREATMENT PRINCIPLE


Accelerate blood circulation to improve regeneration ability of collagen fibers and promote metabolism.

650nm laser and 630nm PDT can enhance the activity of Adenosine Triphosphate (ATP) of subcutaneous tissue, and ATP is the main energy transmission medium between cells which help to encourage the metabolism of the tissue . So 650nm laser and 625nm PDT help to enhance blood flow and make oxygen and nutrients send to the hair follicles, and 808nm laser will help solution penetration , as a result , it achieves the efficacy of halting the progression of hair loss.

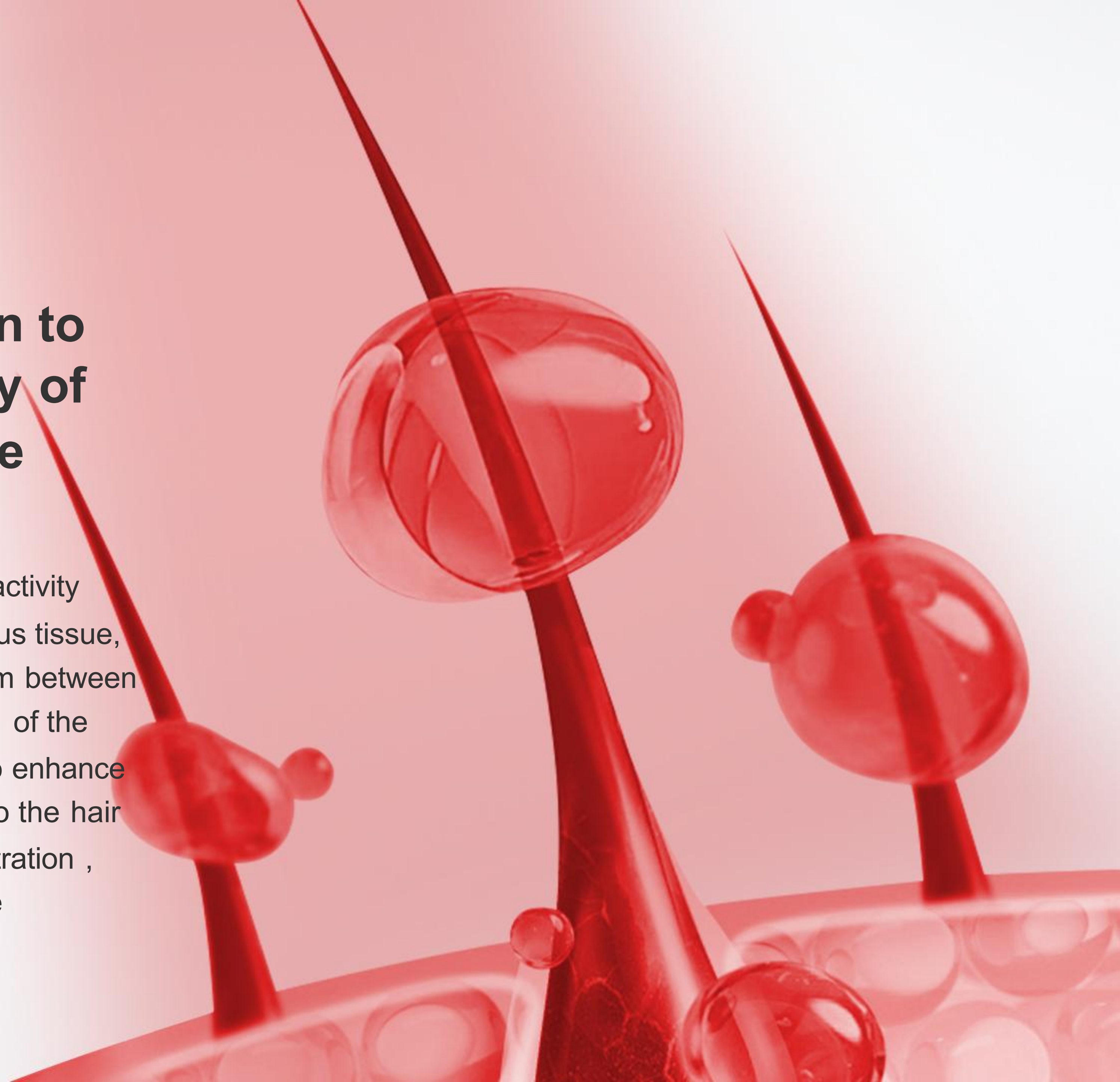


Modulate the hair follicles

SH-650 patented narrow wavelength light and EMS can effectively stimulate hair follicles, and turn 80% of the hair follicles which is in the state of hibernation to active status , and attain the effect of the cessation of hair loss and restoring.

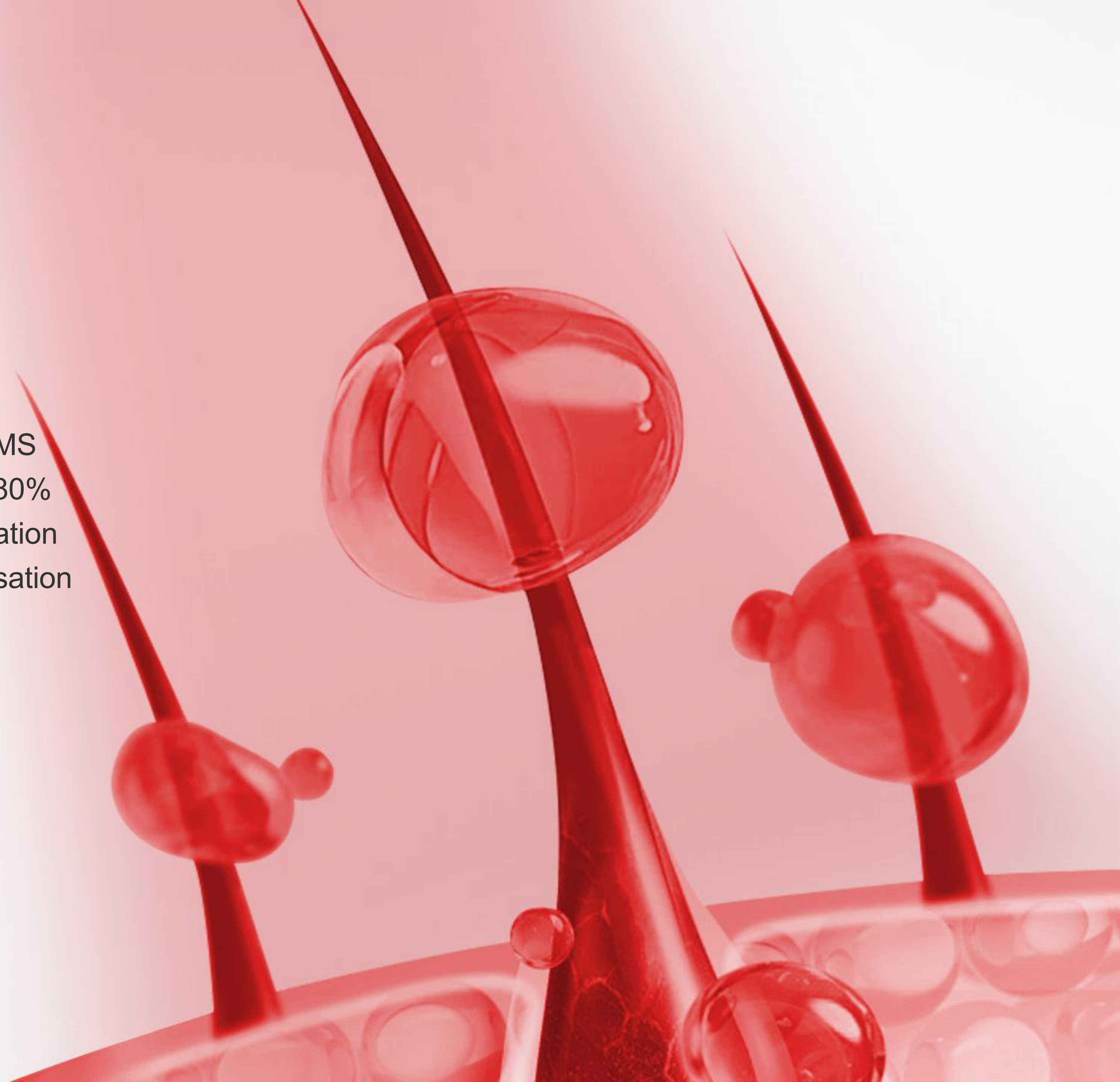

Regulate oil secretions

Excessive oil secretion also is a reason for causing hair loss , Abnormality of sebaceous glands leads to excessive oil secretion, while Dihydro Testosterone (DHT) in the oil released by sebaceous glands will trigger more hair follicles dwindling or even shrinking, then, hair loss is caused. SH-650 Hair Rejuvenation can dredge the sebaceous glands duct to decrease and control oil secretion so as to improve hair condition.

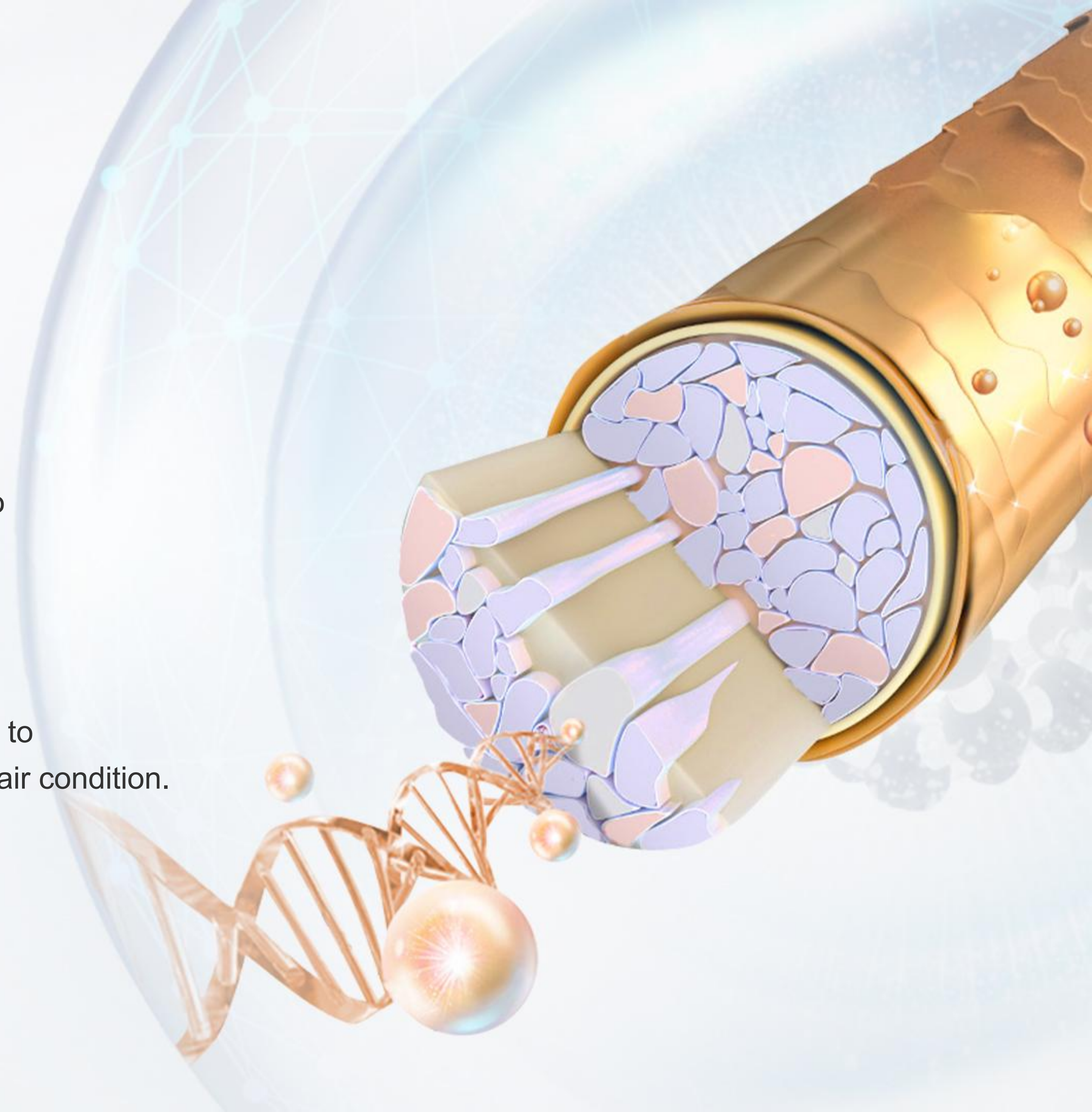




TREATMENT PRINCIPLE

Promote scalp health, improve hair quality

SH-650 composite hair mask irradiates hair follicles to accelerate hair growth, and strengthen the hair elasticity and increase hair density making the hair pitch-black shine.

Promote nutrient absorption

SH-650 is a patented cold light source, which won't hurt hair, it is neither chemical nor drugs but its penetrability is better than them. It overcomes the weakness of various chemicals hair fluid which is unable to fully infiltrate into hair root.

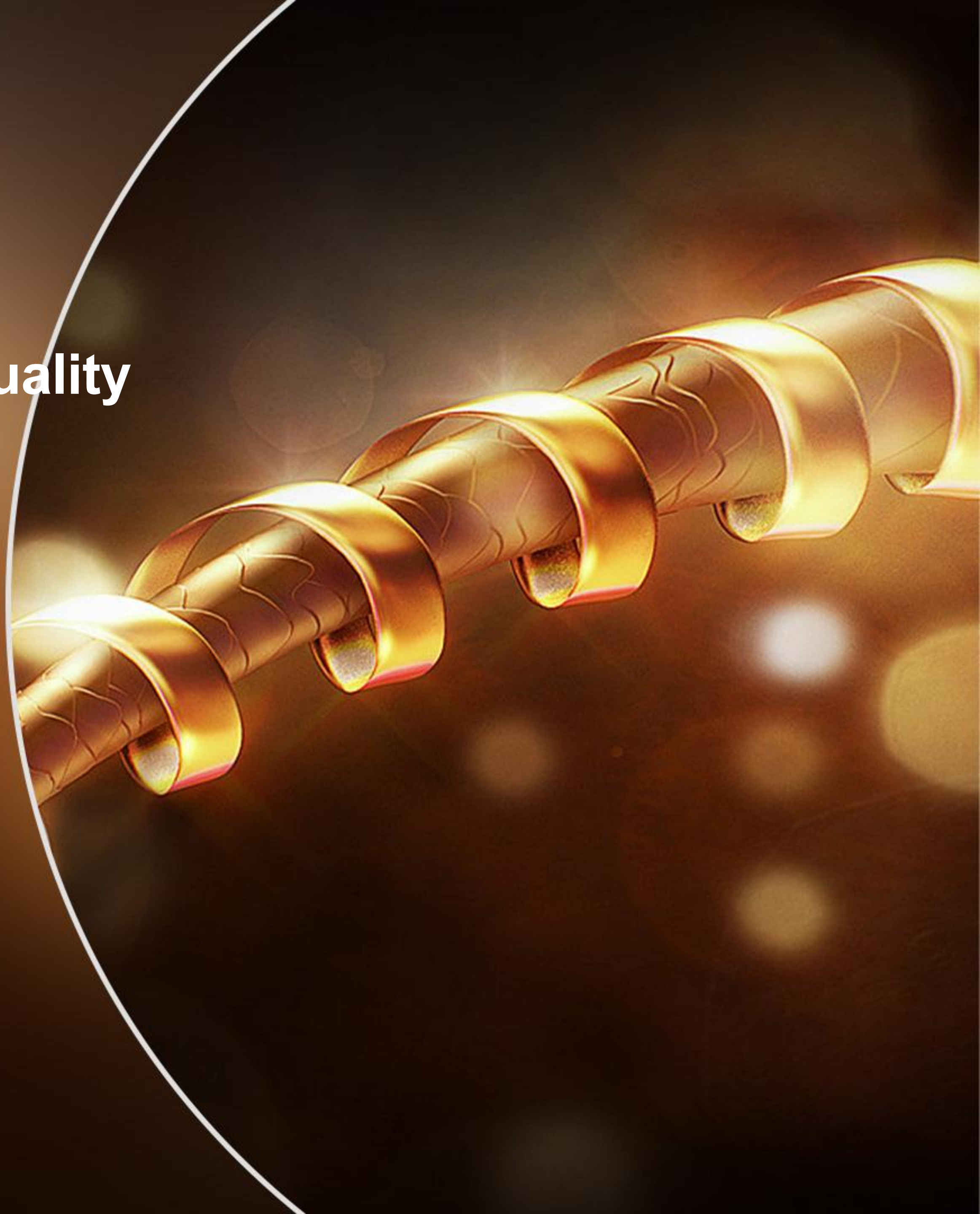



1

1. Wash hair and scalp. If the patient can wash hair and scalp in home, it will save the treatment time.



3

3. Use the oxygen spray pen to clean the scalp with Saline.



5

5. For the scalp can absorb the lotion better, use the Microcurrent massage comb to massage the scalp 5- 8 minute after spraying the minoxidil lotion.



2

2. Use the Hair analyzer to analyse the condition of hair and scalp and store patients' date.



4

4. Use the Medicine Sprayer to spray minoxidil lotion on the treatment area.
5% density for man , 3% for woman.



6

6. Then open the system, set the energy and work time to start to irradiate the scalp. The irradiating time is 30 minutes for 1 time in one week or 60 minutes for 1 time every two weeks.

7. After irradiating finished, the patient need wait 1 hour to wash the hair and clean the mioxidil lotion.

POST TREATMENT CARE

**Every week 1 treatment and 2-3 month is a session.
8-12 treatment is needed for one course.**

**After 1 session, need 1 month interval to start
another session according to the
patient situation as followings:**

Hair loss extent : the severity of hair loss;

Hair growth results: the objective is expected to reach what kind of germinal dense hair effect, such as hair loss area, thinning hair and hair follicles of dormancy level.

The number of germinal stage: laser to germinal instrument the treatment of seborrheic alopecia is a stage treatment for hair growth is more beneficial and more scientific and rational.

Note: For better result of hair loss therapy device, the patient can spray or paint the Minoxidil every 2 days in home, only 1ml on the target area and massage 5-8 min. Waiting for 2 hour, then wash the hair and clean the minoxidil.





BEFORE AND AFTER



Before treatment



2 Month
New hair growth



3 Month
More hair growth and thickness




4 Month
Continue to have new growth



LLLT INDICATIONS

SH650-1

- ▶ Hair Loss therapy and increase Hair intensity for men and women
- ▶ Hair loss caused by endocrine strong
- ▶ Hair loss caused by pathogenic and nerve
- ▶ postpartum hair loss
- ▶ Alopecia areata, and increase the hair intensity and hair care after Hair transplant.

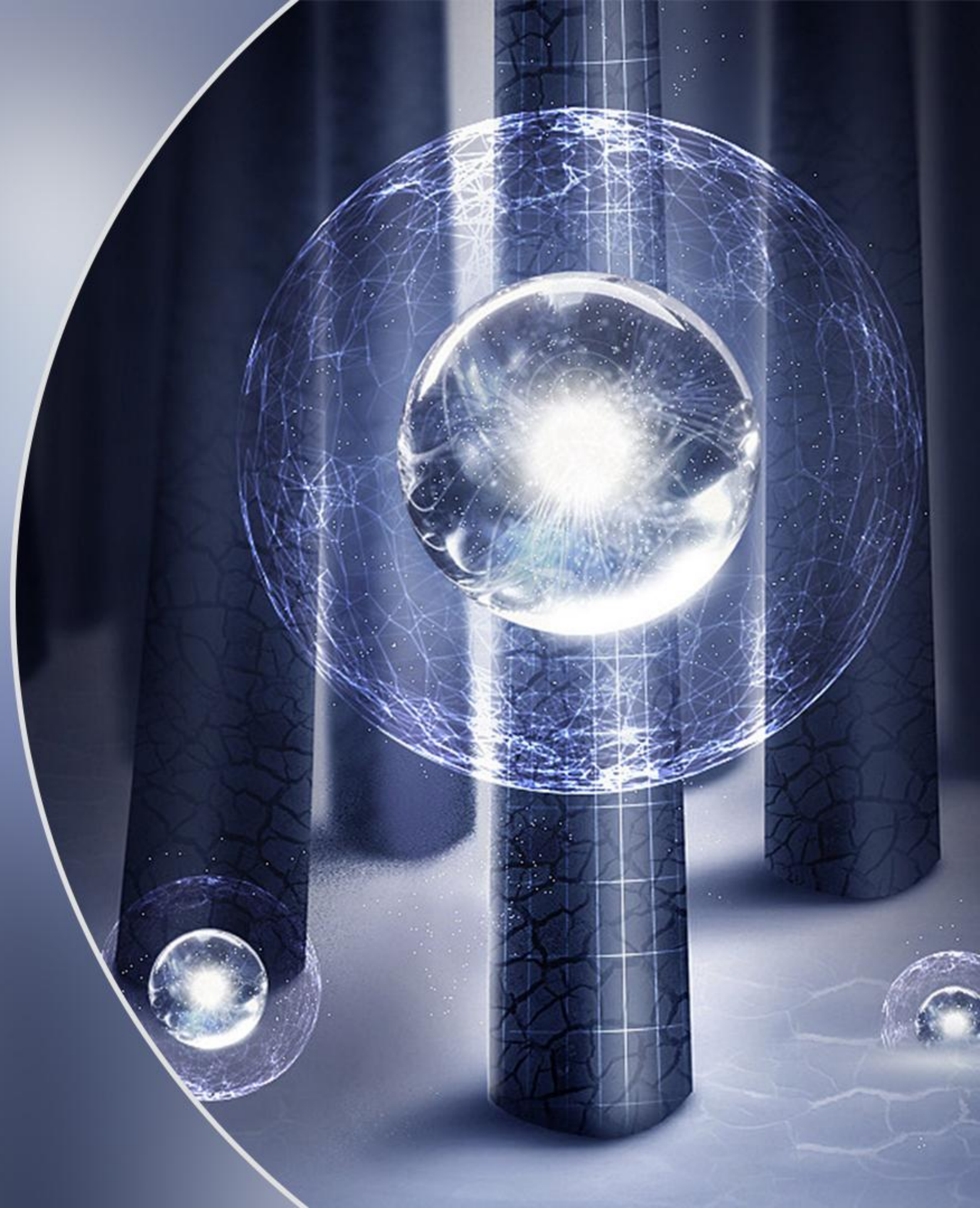

CONTRAINDICATIONS

- ▶ Scalp trauma ;
- ▶ Women's menstrual period, pregnancy, lactation;
- ▶ Not to accept the treatment within one month after the perm;
- ▶ Serious medical illness, such as: cancer, blood diseases, severe diabetes, hypertension, heart disease;
- ▶ If you have a special systemic disease, please inform your doctor truthfully before receiving treatment .





Bei Jing San He Tech Co.,Ltd

Address: Room 204, 2nd Floor, No.7 Building,
Maohuagongchang, NO.1 CAIDA Third Street Nancai
Town, Shunyi District Beijing, China

Website: www.sanhemedical.com

E-mail: admin@sanhemedical.com

Tel: 010-69403826

Fax: +86 10 59777823

Whatsapp: +8618501921559

Name	Low Level Laser Hair Regrowth Machine
Laser output power	650nm diode laser 660mw 808nm IR laser 5600mw
Model	SH650-1
LED output power	630nm Red cool light 1000mw 460nm Blue Cool light 1000mw 580nm Yellow Cool light 2000mw
Treatment Time	0-60min
Pulsing time range	1-10s
Pulsing Delay range	0-10s
Input power	200w
LCD Screen	8inch multi color touch screen
Device Dimension	1630*550*500 mm
Package Dimension	1650*580*580 mm
Rated input voltage	AC230 10%,50Hz 1Hz AC110V 10%,60Hz 1Hz (optional)

